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COVID-Flu RQ Kit Manual

Autumn 2025, Version 1.0

For Real-Time RT-PCR Detection of SARS-CoV-2
and Human Influenza A/B Virus RNA
For Research Use Only

 24 (Cat# COVID-FluRQ24)

 48 (Cat# COVID-FluRQ48)

 96 (Cat# COVID-FluRQ96)

 NG-WI-ASL-72-100

RUO

NovinGene ParsVira

No. 5, Iranshahr St, Tehran, Iran 1581633336.

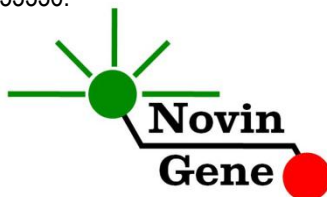
Table of Contents

1. Introduction	3
2. Intended Use	3
3. Background Information	3
4. Test Principle	4
5. Kit Contents	4
6. Packaging models	4
7. Storage and Stability	4
8. Product Use Limitations	4
9. Additionally Required Materials	5
10. General Precautions	5
11. Specimen, Storage and Transport	6
12. Internal Control (IC)	6
13. RNA Isolation	7
14. RT-PCR Protocol	7
15. Devices and software	8
16. Programming of the Rotor-Gene	8
17. Programming StepOne and Other Machines	9
18. Data Analysis: Rotor-Gene	9
19. Data Analysis: StepOne	12

20. Sensitivity.....	15
21. Disposal Method	15
22. Technical Support.....	15
23. Contact Information.....	15
24. References	15
25. Symbols.....	16

1. Introduction

COVID-Flu RQ kit provides a ready-to-use Real-Time RT-polymerase chain reaction (PCR) test designed for detecting SARS-CoV-2 and Influenza A or Influenza B virus. All required reagents are included in the PCR Mix provided in the kit. This Mix also contains a different series of primers and probe for detecting a synthetic DNA sequence. The kit supplies this synthetic DNA sequence as an Internal Control (IC). The IC can be used either during DNA extraction or in the PCR reaction to prevent false negative results due to failure in the above steps.

This kit is intended for Research Use Only!

2. Intended Use

COVID-Flu RQ kit provides a ready-to-use One-Step Multiplex Real-Time RT-PCR system for the detection of SARS-CoV-2 Influenza A virus or Influenza B virus (without differentiation Influenza A and B) with Rotor-Gene, StepOne and MIC.

3. Background Information

Coronaviruses are enveloped viruses with single stranded RNA genome. Up to 2002 they were considered mostly as causes of common cold. However, with the emergence of SARS (Sever Acute Respiratory Syndrome by SARS CoV) in 2002, then MERS (Middle East Respiratory Syndrome by MERS CoV), and recently COVID-19 (by SARS-CoV-2), they are considered major health problem issues.

Human influenza A and B are enveloped RNA viruses of the Orthomyxoviridae family. While both viruses cause seasonal flu epidemics, Influenza A virus is responsible for the majority of them and may also cause pandemics.

4. Test Principle

The pathogen is detected using PCR, where primers specific to the target genome amplify its unique sequence. Real-Time PCR facilitates the detection of the amplified product through fluorescent-labeled probes. Therefore, monitoring fluorescence provides a means for detecting the target without requiring post-amplification analysis. This eliminates the possibility of PCR product contamination.

5. Kit Contents

The kit contains a manual a flash card and the following reagents:

Label	Content	Quantity
CoV-Flu Mix	RT-PCR Mix*	360 µl
Resp A Pos Ctrl	Positive Control	150 µl
Internal Ctrl	Internal Control*	250 µl
Water	PCR Grade Water	200 µl

* 1, 2 and 4 tubes for 24, 48 and 96 reaction kits respectively.

6. Packaging models

The kit is available in 24, 48, and 96 reactions of 25 microliters.

7. Storage and Stability

The kit components should be shipped and stored at -20°C and are stable until the expiration date mentioned. Avoid repeated freeze-thaw more than three times to prevent reduced sensitivity.

8. Product Use Limitations

- This kit is intended to be used only by specially instructed and trained personnel.

- The user manual should be strictly followed, and any modification will invalidate the results.
- The kit and its contents should not be used past the expiration date on the package.
- The kit and its contents should not be used if there is any sign of pink or red color on the Warm Mark label.
- This kit is for Research use only and is not validated for IVD (in vitro diagnostics) applications.

9. Additionally Required Materials

To use this kit, you need the following items:

- Real-Time PCR machine and the accessory computer
- Tabletop microtube centrifuge
- Vortex Mixer
- Dry Block Heater
- Adjustable pipettors and nuclease free filtered tips
- RNA extraction kit and required equipments/items
- PCR microtubes
- Disposable powder-free gloves
- Cold block

This kit does not require cDNA synthesis reagents!

10. General Precautions

To prevent false results, always pay attention to the following points:

- **Treat all samples as potentially infectious.**
- Within the pre-PCR work area assign three separate spaces for: a) Sample storage and extraction, b) Reagent preparation where the master-mix is aliquoted into tubes and c) Reaction preparation area for addition of extracted RNA to the tubes.

- Always wipe the working surfaces with 70% Ethanol before and after work.
- Thaw on ice kit components completely, mix by flickering followed by a quick spin and store on crushed ice after.
- Keep RT-PCR Mix tube at -20°C at all times. Take it out just before use and return it to freezer immediately after.
- Do not place PCR tubes on crushed ice. Use cold block instead.

11. Specimen, Storage and Transport

We recommend upper respiratory tract samples mainly nasopharyngeal swabs and, in some cases, lower respiratory samples including sputum, endotracheal aspirate or bronchoalveolar lavage. Samples can be stored at 2-8°C for a few hours or at -20°C or lower for up to a few days.

12. Internal Control (IC)

To assess the possibility of RNA extraction failure and PCR inhibition and prevent false negative results, the COVID-Flu RQ kit contains an internal control (IC). This IC can be used during the extraction process or added directly to the CoV-Flu Mix. To monitor both RNA extraction and PCR reaction, the IC should be added to the mixture of lysis buffer and patient sample. The required volume of IC is 10% of the elution buffer. For instance, if the extracted RNA is eluted with 100ul, then 10ul of IC should be added to the mixture of lysis buffer and patient sample. **Please note that the IC should not be added directly to the patient sample (i.e, before the addition of lysis buffer) as it loses its efficiency.**

If the IC is added to CoV-Flu Mix, only PCR inhibition can be monitored. For this purpose, 1ul of the IC should be added to each

reaction. For example, for 10 PCR reactions, 10ul of the IC should be added to 150ul of CoV-Flu Mix before it is added to the tubes. In a successful RNA extraction and PCR test, the IC should generate a CT of 27-35 in the ROX/Orange Channel.

13. RNA Isolation

RNA isolation can be performed using different kits from various manufacturers. We recommend the following:

- High Pure Viral Nucleic Acid Kit (Cat. no. 11858874001, Roche Applied Science, Mannheim, Germany)
- QIAamp Viral RNA Mini Kit (Cat. no. 52904, Qiagen GmbH, Hilden, Germany)
- QIAampUltraSens ® Virus Kit (Cat. no. 53704, Qiagen GmbH, Hilden, Germany)
- QIAampMiniElute Virus Spin Kit (Cat. no. 57704, Qiagen GmbH, Hilden, Germany).

14. RT-PCR Protocol

Thaw the reagents on ice completely, followed by a brief mixing and a quick spin. Place the required number of tubes on a cold block. Consider one tube for each sample plus one for positive sample and one for the negative control or water.

If the IC is introduced during the extraction process, pipette 15µl of CoV-Flu Mix directly to each PCR tube.

If the IC is added to the CoV-Flu Mix, add 15ul of the prepared mix (as described in section 12) to each PCR tube.

Then add 5ul of extracted RNA, Pos Ctrl, or water to each tube.

Cap the tubes and visually inspect to ensure all are capped securely. Place tubes in the machine.

Note: If using Rotor-Gene attach the locking ring too.

15. Devices and software


COVID-Flu RQ kit is designed to work with Rotor-Gene, StepOne and MIC.

16. Programming of the Rotor-Gene

Before you start the machine, make sure you have attached the locking ring on the rotor!

Open the COVID-Flu template file for Rotor-Gene (provided in the flash card, or accessible by kit QR code); COVID-Flu 0.1 is for strip tubes and COVID-Flu 0.2 is for 0.2ml tubes. Program starts.

Note: For Gain Optimisation, in the View menu select the Gain Optimisation. Adjust the setting according to the following image. Make sure to set the Tube Position to number 1 for all channels (note that Tube number 1 should contain CoV-Flu Mix).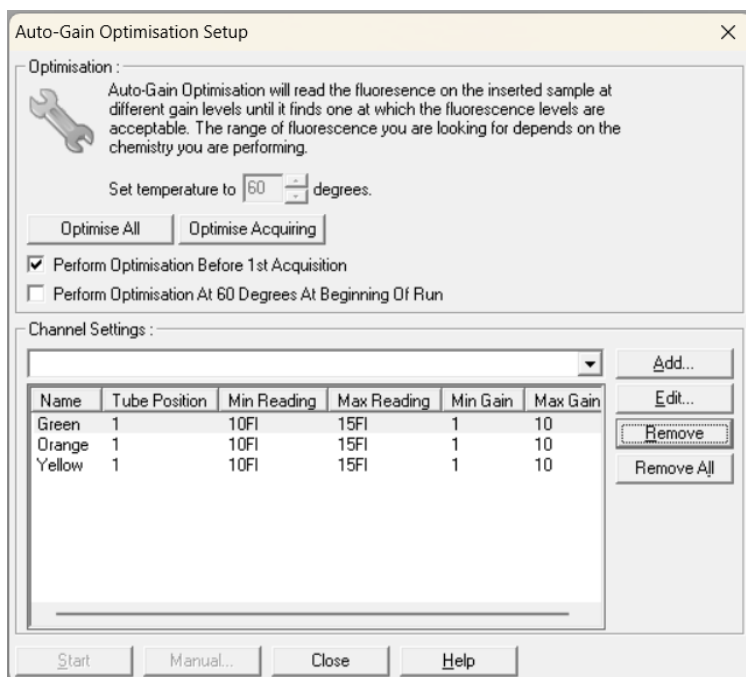

Click on Start button (Green button on the top menu). On the pop-up window click start again and save program on desired location. You may also set the machine according to the following table.

Step	Temperature and time	Cycles
1	50°C x 10 min	1
2	95°C x 3 min	1
3	95°C x 5 sec	45
	60°C x 30 sec	

Fluorescence should be collected at 60°C via Green, Yellow, Orange and channels.

17. Programming StepOne and Other Machines

If you apply this kit to other Real-Time PCR machines, program it according to the following table (please note that following program is not applicable to Rotor-Gene):

Step	Temperature and time	Cycles
1	95°C x 10 min	1
2	95°C x 3 min	1
3	95°C x 5 sec	45
	60°C x 40 sec	

Fluorescence should be collected at 60°C for FAM, VIC and ROX dyes.

Please note that CoV-Flu Mix does not contain ROX dye as a normalizer!

18. Data Analysis: Rotor-Gene

Before analyzing results, make sure that in the sample menu, Patient samples are defined as "unknown" and Negative control or

no template control as "Negative Control" or "NTC" respectively. To analyze data briefly, click on the Analysis menu and then under Quantitation tab double-click on Cycling the A. Green. Manually put threshold at 0.1. Repeat the above for the Yellow and the Orange Channels.

Figures 1, 2, 3 represent typical graphs for the Rotor-Gene.

To interpret the results, please note that:

- Signal in the **Green** channel is due to **SARS-CoV-2**, the **Yellow** channel is due to **Influenza A** and **B** and the signal in the **Orange** channel is due to **IC**.

Note that a sample is considered Positive only if it has a sigmoid graph and log phase, and only then CT is reliable and can be used. In the absence of a sigmoid graph and log phase, the sample is considered Negative, and CT, if present, is not reliable.

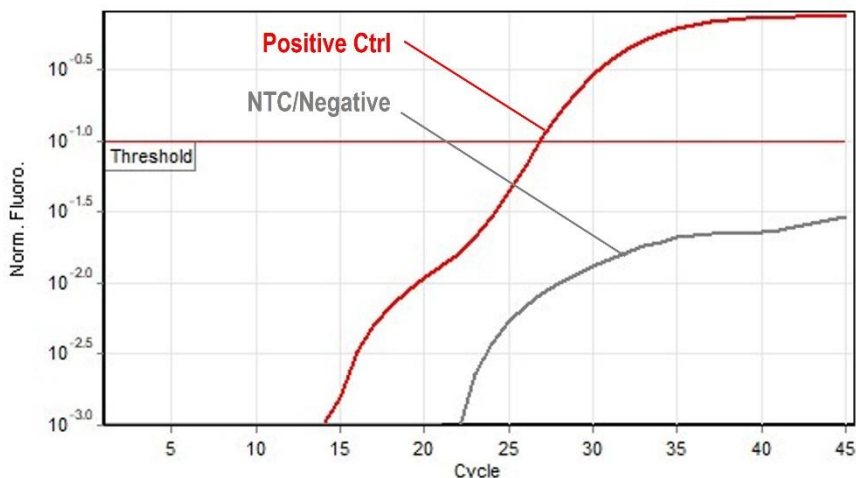
Fig 1. Typical Control graph in Green channel for Rotor-Gene

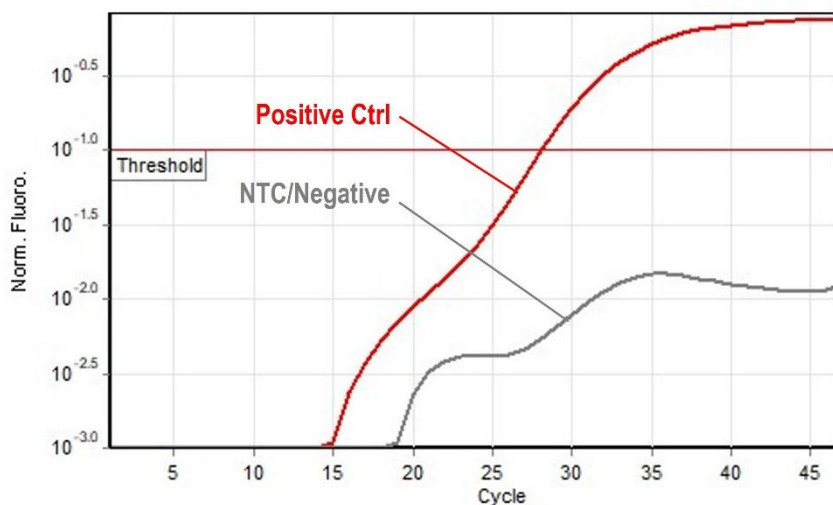
Fig 2. Typical Controls graph in Yellow channel for Rotor-Gene

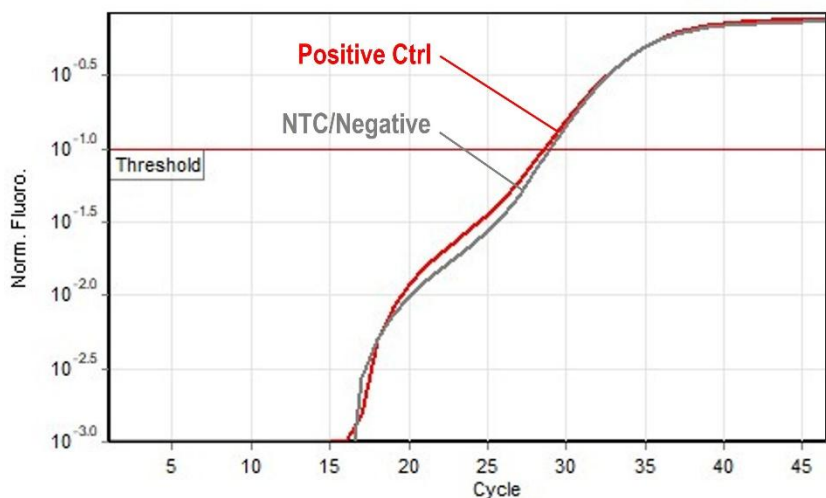
Fig 3. Typical Internal Control graph in Orange channel for Rotor-Gene

Consider the following points when analyzing:

- A sample is **Positive** for SARS-CoV-2 if it is positive in the **Green** channel with sigmoid graphs and CT of less than 40.
- A sample is **Positive** for Influenza A or B if it is positive in the **Yellow** channel with sigmoid graphs and CT of less than 40.
- A sample is **Negative** for SARS-CoV-2, Influenza A, B if it is negative in the Green and Yellow channels while it is positive in Orange channel with a sigmoid graph and CT of 20-35.
- Results are **Inconclusive** and the test should be repeated if a sample is negative in all channels.

Interpretation of results is summarized in the following table.

Green	Yellow	Orange	Result
+	-	+/-	Positive for SARS-CoV-2
-	+	+/-	Positive for Flu A/B
-	-	+	Negative
-	-	-	Inconclusive

19. Data Analysis: StepOne

Analyze data according to the StepOne Manual. Briefly, click on Analyze and set the threshold for the **SARS-CoV-2/FAM** at 1000 and repeat it for **Influenza/VIC** channel and put the threshold at 300 for the **IC/Orange** channel.

Figures 4, 5 and 6 represent typical graphs for the StepOne machine.

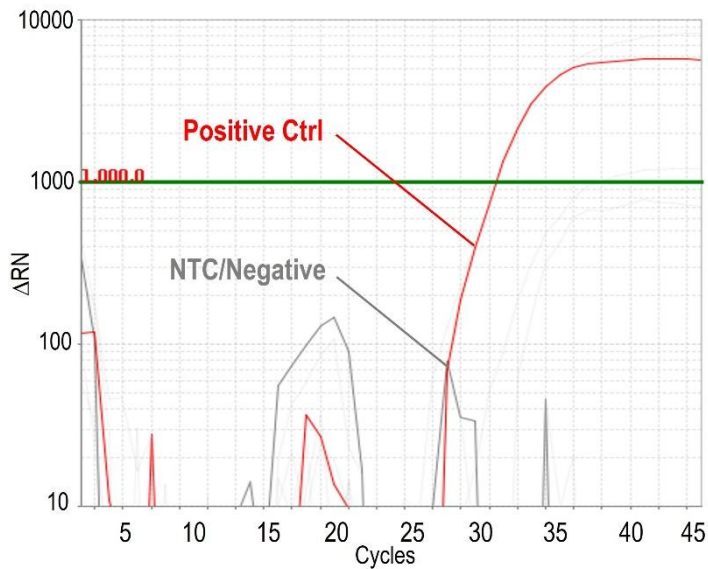
Fig 4. Typical Controls graph in FAM channel for StepOne

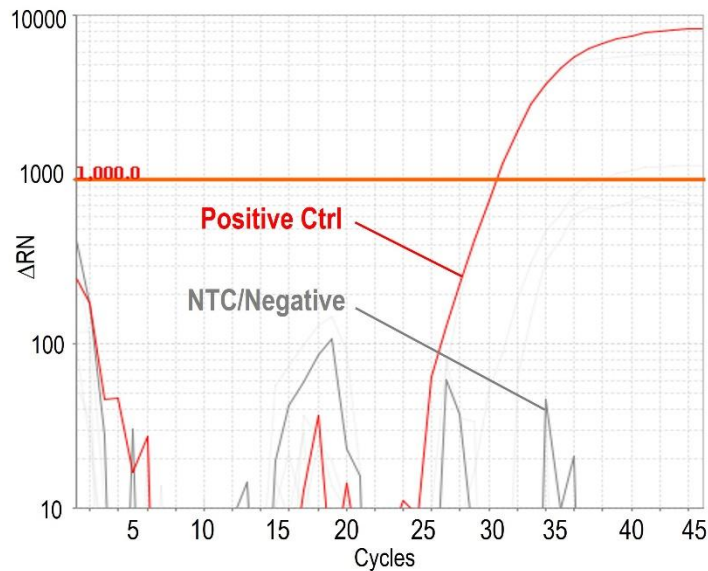
Fig 5. Typical Controls graph in VIC channel for StepOne

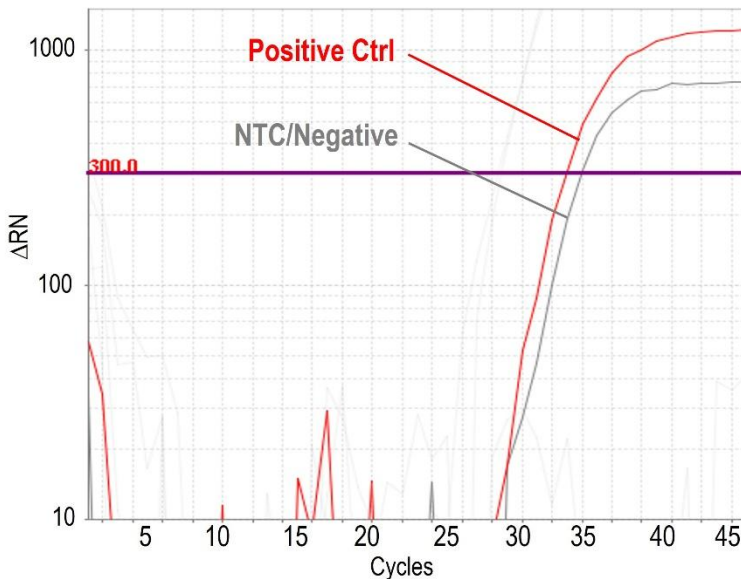
Fig 6. Typical IC graph in ROX channel for StepOne

Consider the following points when analyzing:

- A sample is **Positive** for SARS-CoV-2 if it is positive in the **FAM/SARS-CoV-2** channel with a sigmoid graph and a CT of less than 40.
- A sample is **Positive** for Influenza A or B if it is positive in the **VIC/Influenza A,B** channel with a sigmoid graph and a CT of less than 40.
- A sample is **Negative** if it is negative in the FAM/SARS-CoV-2 and VIC/Influenza A,B channels while it is positive in the **IC/Orange** channel with a sigmoid graph and CT of 27-35.
- Results are **Inconclusive** and the test should be repeated if a sample is negative in all channels, the FAM/SARS-CoV2 and the VIC/Influenza A,B and the **ROX/IC** channels.

The interpretation of results is summarized in the table on page 12.

20. Sensitivity

The analytical detection limit of the kit was assessed with a cloned target of SARS-CoV-2 and Influenza A and B genome and showed a limit of detection equal to 17 copies/μl for SARS-CoV-2, 10 copies/μl for Influenza A and Influenza B.

21. Disposal Method

The contents of the kit do not require any special treatment before disposal and can be directly discarded. Infectious specimens should be maintained in 5% Sodium Hypochlorite overnight and then discarded.

22. Technical Support

For technical support, contact us via

Phone: +98 993-6223241

Email: info@novingene.com

23. Contact Information

NovinGene ParsVira

Address: No. 5, Iranshahr St, Tehran, Iran 1581633336.

Tel: +98 21-88837393

+98 990-1813124

Email: info@novingene.com



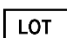
Website: www.novingene.com

24. References

- Hu, B., Guo, H., Zhou, P. and Shi, Z.L., 2021. Characteristics of SARS-CoV-2 and COVID-19. Nature reviews microbiology, 19(3), pp.141-154.

- Lamers, M.M. and Haagmans, B.L., 2022. SARS-CoV-2 pathogenesis. Nature reviews microbiology, 20(5), pp.270-284.
- Mackay IM. Real-time PCR in the microbiology laboratory. Clin. Microbiol. Infect. 2004; 10 (3): 190 – 212.
- Palese, P. and Kingsbury, D.W., 2011. Genetics of Influenza Viruses. Springer.
- Paules, C. and Subbarao, K., 2017. 'Influenza', The Lancet, 390(10095), pp. 697–708.
- Uyeki, T. M., Hui, D. S., Zambon, M., Wentworth, D. E., & Monto, A. S., 2022. Influenza. Lancet (London, England), 400(10353), 693–706.

25. Symbols

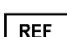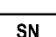 RUO Research use only	 Manufacturer	 Consult instructions for use
 LOT Lot number	 Content sufficient for <n> tests	 Use-by date
 REF Catalogue number	 SN Serial number	 Temperature limit

For more information and resources please visit our website; www.novingene.com